


Technical Note no. : 0902
Date : 2009-01-23
Instrument : OtoAccess ver. 1.01
Subject : Installing on XP with SP3



Installing OtoAccess 1.0.1 on a Windows XP SP3 (NON English).

When installing the SQL Server Part during a Standalone or Server installation a Message box pops up during the installation.

Saying:

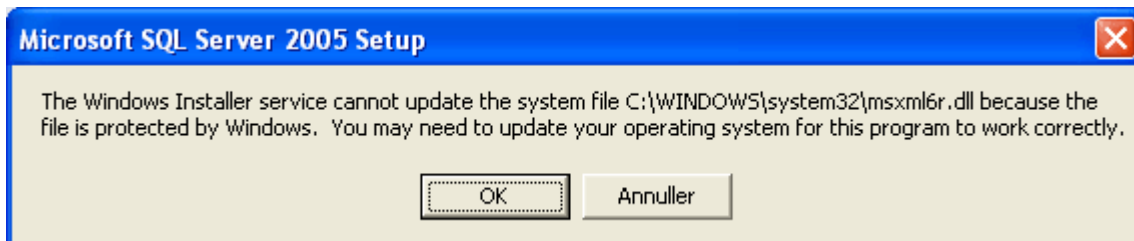

This is caused by:

The problem is that Windows places the msxml6 under System File Protection for SP3, and this change conflicts with the pre-existing requirements of installing SQL Server. During installation msxml64.dll attempts to make a language change that is not allowed under System File Protection.

The installation is not affected since msxml6 is distributed with SP3.

Necessary action: Click OK

The installation is not affected.