

Technical Note no. : 0906
Date : 2009-06-04
Instrument : Eclipse
Subject : ABRIS important update



Description:

It has been found that on high end computers (processor speed) ABRIS can occasionally fail to initialize sound when starting the recording – this is causing the recording to be run without the stimuli being presented.

This has been fixed in ABRIS 1.05 build 1, which can be downloaded through the dealer's web login or ordered on a CD from Interacoustics.

Because it is impossible to determine where the system starts to fail, we therefore recommend -

Updating ABRIS no matter what earlier version of ABRIS or Computer is used.

Login for dealers:

<http://extranet.interacoustics.com/Extranet/TechnicalSupport/Software.asp>

Solution:

1. Use Windows control panel "Add/Remove" to remove any previous version of ABRIS (data and setting will not be lost).
2. Install ABRIS 1.05 build 1