



Science **made** smarter

Technical Release Note  
ONLY for Distributors

# AD629 Firmware 1.41.1



**Interacoustics**



Title:	AD629 Firmware 1.41.1 Technical Release Note
Project:	AD629 Firmware 1.41.1
Author:	Renskje K. Hietkamp – Clinical Product Manager Lillian Bjørn Nielsen – Senior Product Manager
Date issued:	June 2022

## Table of Contents

Introduction .....	2
Improved stability .....	2
Compatible software .....	2
How to update .....	2
Overview of improvements .....	3



## Introduction

Dear distributor and colleague,

Interacoustics is happy to announce the release of the new **AD629 Firmware 1.41.1**. This release is a service update.

Please note that this update is **only available for AD629 v2**.

This new firmware is not compatible with AD629 legacy instruments or any other instrument.

The aim of this technical release note is to provide you with the necessary details for the update process. This should ensure that you are able to assist and guide the user in the best possible way, reducing the risk of errors after the update.

This release note is only for you! Please do not forward it to end-users.

## Improved stability

This service update offers improved stability when performing speech audiometry using built-in speech material with the AD629 v2 in hybrid mode.

## Compatible software

The AD629 v2 firmware version is 1.41.1. It is compatible with

- Diagnostic Suite 2.9 and 2.10
- Noah 4.12.1
- OtoAccess® Database 2.0 and 2.1

## How to update

The new firmware is available on the Interacoustics extranet:

<https://www.interacoustics.com/downloads/ad629-v2/ad629-v2-product-updates>.

1. Download the 'AD629 v2 firmware 1.41.1' zip files to your computer.
2. Unzip the firmware archive on a new FAT32 formatted USB stick.
3. Insert the USB stick in the unpowered AD629 v2.
4. Turn on the instrument and wait until you see the Tone test display.
5. Press 'Setup' and choose 'Instrument settings'.
6. Press the function key named 'Install' and follow the instructions on the instrument display.
7. Wait for the instrument to return to the Tone test display.  
**Do not shut down the instrument or remove the USB stick while updating!**
8. Always restart the instrument after an update before using it.

Optional: Press Shift+Setup to check that the new firmware has been installed on the instrument.



## Overview of improvements

Description	INC
AD629 v2 Hybrid mode issue	INC63528 INC64284