



Attention

Filling the water reservoir



Caution

Do **not** attach the power cord or turn on the irrigator until the reservoir has been properly filled with **distilled water**. The Air Fx will be shipped from the factory without **distilled water** in the reservoir. The distilled water must be added, and the pump primed before applying power. Failure to maintain the proper level of distilled water can result in an inability to attain the preset cool temperature and a resultant error message on the display. Running the unit with insufficient water in the reservoir may also damage the coolant pump. Misuse is not covered under warranty.

1. Check that the irrigator is powered OFF.
2. Remove the cap to the water reservoir on the top of the irrigator. Do not force the cap or use mechanical tools to remove the cap.
3. Fill the reservoir with distilled water up to 1cm from the bottom of the cap. Take care not to spill any water into the unit except into the water reservoir to prevent damage to the circuitry and prevent any electrical shock to the user. If any water is spilled, wipe it up immediately.
4. Purge air from the hoses, pump and handle by connecting the included **60cc syringe & tube** to the Fill / Drain Water Outlet. Fully push the tube into the Fill / Drain Water push-in connector (about 1" (2cm) of tubing). Gently pull on the tubing to confirm the tube is secure.
5. Switch the Fill / Drain Valve to the Fill / Drain position.
6. Draw back the syringe to draw the air out of the internal tubing and draw distilled water into the syringe. If there is a train of air bubbles or there is no suction, then set the Fill / Drain Valve to the Run position and reconnect the tube. Once reconnected, set the Fill / Drain Valve to the Fill / Drain position.
7. Disconnect the syringe from the tube and keep the tube held above the irrigator to prevent the water from exiting the tube. Squirt any distilled water drawn back into the reservoir. Reconnect the syringe to the tube.
8. Repeat steps 6 to 7 two more times to draw the remaining air out of the lines.
9. Set the Fill / Drain Valve to the Run position.
10. Disconnect the syringe. The tube can be removed by pushing on the sides of the tubing connector's gray collar around the tubing while simultaneously pulling on the tubing.
11. Place the cap back on the water reservoir.
12. Attach the AC power cord to the back of the irrigator and switch on the Air Fx using the power switch next to the power cord. Water will automatically be pumped through the hoses into the handle and returned to the reservoir.
13. Test system by running irrigator on the warm cycle then the cool cycle. If the irrigator is able to do this, then it is ready to use. If the irrigator is unable to reach the desired temperature, then repeat the fill instructions.



Figure 1 Air Fx Rear Panel