




Attention

Initial filling of the irrigator

READ THE ENTIRE SHEET FIRST AND THEN FOLLOW THE STEPS IN ORDER

The Aqua Stim will be shipped from the Micromedical factory without any water in the system. Please fill according to the instructions of the manufacturer. The Aqua Stim cannot be used without adequate water in the water reservoir. The water must be added before applying power. Failure to ensure the proper installation can result in damage to the unit.

1. Read the User's Manual and set up irrigator with tanks and hoses according to setup instructions.
2. Fill the external water tank with water up to the fill line on the tank
3. Turn on irrigator
4. Select Cool

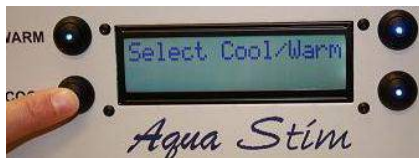


Observe Next View >




IF either of the errors below occurs



OR

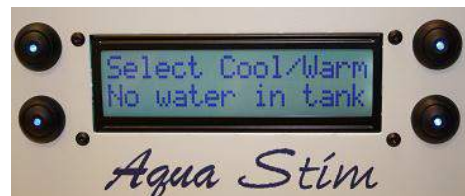

There is a bubble stuck in the internal tank. Turn off power then power back on, select COOL and tilt the irrigator to a **30 degree** angle and hold for ONLY 4-5 seconds during FILLING cycle. Return the irrigator to the flat level surface, watch for bubbles exiting the irrigator going to the external tank in the label gray tube. That indicates the trapped bubbles have been released and the irrigator is ready for use. If errors still persist, please call Micromedical Technical Support at (217) 483-2122 x142

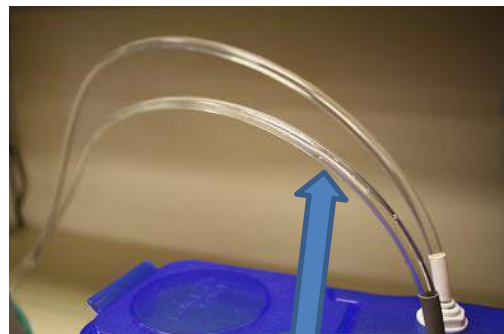

Watch for bubbles in this tube