



Science **made** smarter

Technical Release Note
ONLY for Distributors

Diagnostic Suite 2.10



Interacoustics



Title:	Diagnostic Suite 2.10 Technical Release Note
Project:	Diagnostic Suite 2.10
Author:	Helle Jensen – Clinical Product Manager Lillian Bjørn Nielsen – Senior Product Manager
Date issued:	December 2021

Table of Contents

Introduction.....	2
Windows® requirements.....	2
Diagnostic Suite 2.10 – support of legacy products.....	2
New features in Diagnostic Suite 2.10	2
Getting the installation files	3
From the extranet.....	3
From USB	4
Diagnostic Suite upgrade.....	4
Additional improvements.....	4



Introduction

Dear distributor and colleague,

Interacoustics is happy to announce the release of the new **Diagnostic Suite 2.10**. This release is a smaller update for Diagnostic Suite with some small improvements and minor features. No new firmware updates for any devices.

This release note is only for you! Please do not forward it to end-users.


The aim of this technical release note is to provide you with the necessary details for the update process. This should ensure that you are able to assist and guide the user in the best possible way, reducing the risk of errors after the update.

Windows® requirements

Before installation, please ensure that the Windows® operating system is compatible with the new software module.

	Diagnostic Suite	Noah 4	OtoAccess® Database
Current released version	2.10	4.12.1	2.1
Windows® compatibility			
7 Professional – 32 bit	Yes	Yes	Yes
7 Professional – 64 bit	Yes	Yes	Yes
8 Professional – 64 bit	Yes	Yes	Yes
10 Professional – 32 bit	Yes	Yes	Yes
10 Professional – 64 bit	Yes	Yes	Yes

Diagnostic Suite 2.10 – support of legacy products

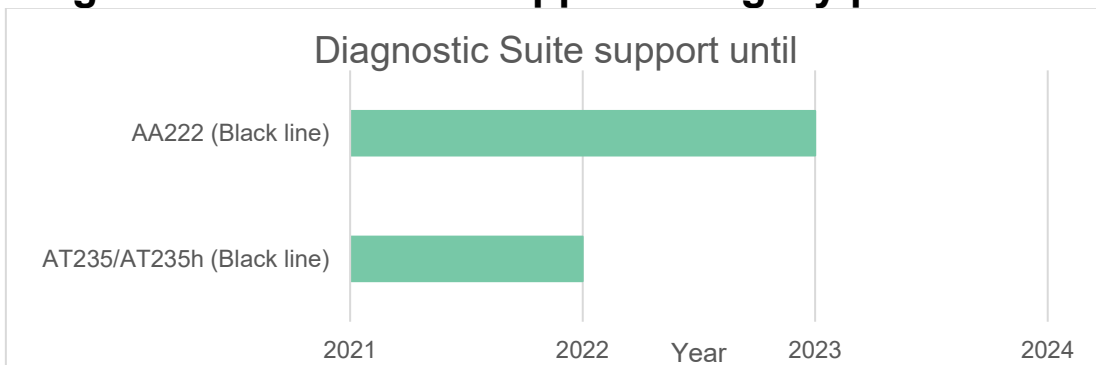


New features in Diagnostic Suite 2.10

All audiometric tests will be visible in the test menu even if there is no license for the specific test. This makes it possible to view data from a specific test in Diagnostic Suite without the license for that test. Unlicensed tests are view only.



Another new feature is a dB HL bar under the graph when doing free-field speech testing. The bar can only be activated when IEC is set as the speech standard. It can be activated under Menu – Setup – General setup – Speech settings.

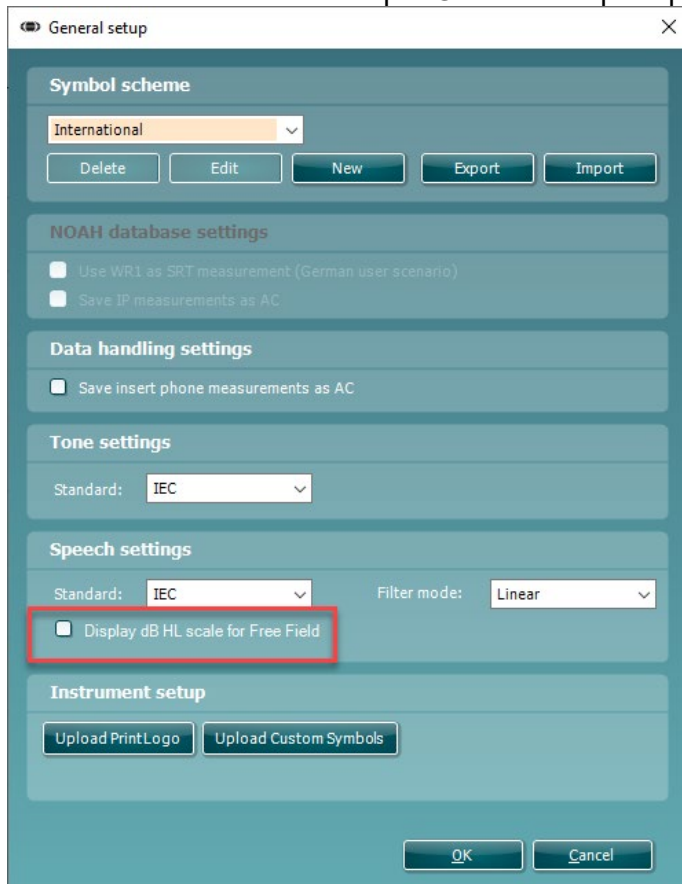

Getting the installation files

When updating a previous version of Diagnostic Suite, you have two options: Update from the Interacoustics extranet or directly from a USB.

From this release, the Diagnostic Suite upgrade package has been split up into Diagnostic Suite upgrade and Diagnostic Suite calibration tool. In the Diagnostic Suite upgrade, the content is only the latest version of the Diagnostic Suite. This is made to make the coming releases smoother and make each device independent of the others.

From the extranet

If you have downloaded the Diagnostic Suite upgrade and Diagnostic Suite calibration tool, do not uninstall the previous version.

Diagnostic Suite upgrade: Includes the upgraded version of Diagnostic Suite. This means that you must have a previous version of Diagnostic Suite installed on the PC before you can install this package.

Diagnostic Suite calibration tool: This package includes the software and firmware tools needed to calibrate/upgrade all instruments supported by Diagnostic Suite.



From USB

If you install from a USB, you must uninstall the previous version before installing the newer version.

Remember to uninstall both installations in add remove programs/Software/Apps & features:

Diagnostic Suite	Interacoustics A/S
Diagnostic Suite Setup	Interacoustics A/S

Proceed to hardware requirements when the above has been clarified with the customer.

In case you need to install Diagnostic Suite for the first time on a PC, contact Customer Service to order Diagnostic Suite on a USB.

Diagnostic Suite upgrade

1. Open the Diagnostic Suite folder and start the installation of the Diagnostic Suite software by pressing the setup.exe file. Please note that administrator rights are required to perform this installation.
2. Follow the steps in the installation wizard.
3. This version can in some cases cause the PC to restart during installation. This will happen on Windows® systems that are not fully updated according to Windows® updater. Should this happen, please make sure that the latest version of .NET framework is installed before running the installation again.

Additional improvements

INC	Title
INC-47687	Diagnostic Suite 2.7.2 - Binaural disappears when switching back and forth between Free field 1 and Free field 2 when the system is Setup for 1 speaker line output
INC-50371	[DS] AC40 with Sync Mode - "No Resp" from the audiometer is not transferred as a NR symbol in Diagnostic Suite v.2.7 / Table View
INC-50960	auto masking turns on masking at all intensities for BC.
INC-53229	Settings in Diagnostic Suite Report Editor is not changed, and default template is not hold after close down.
INC-51998	[All devices] Polish speech material is incorrect.
INC-53470, INC-53478	Diagnostic Suite v. 2.8, Affinity Suite 2.6- Font size (e.g., 100 or 125%) affects the print layout size on the printout.
INC-53438, INC-59425	Diagnostic Suite v. 2.8 - Printing out test result from selecting a saved session, by using Native PDF, A lot of unnecessary letters appear in Report field.